

TOOLS TO KEEP SMILING

FKG

FKG Sensipast

*Paste-filler with automatic disengagement!
Efficient, safe and economical!*

Sensitive and intelligent!

Sensipast knows its capabilities and never surpasses them!

When its **active part is blocked**, the **automatic clutch reacts** avoiding the wire rupture and abnormal metal fatigue.

You can **work stressless**, leading the *Sensipast* more deeply into the canal without fear of rupture, for a **perfect obturation!**

...Clutch on board



SENSIPAST


FKG DENTAIRE
Swiss Dental Products



FKG REF. - 99.180.10.01A.AN

Sensitive and intelligent, *Sensipast* is the sole paste-filler with a safety automatic clutch system which knows its capabilities and never surpasses them!

When its active part is blocked or suddenly jammed in a very curved canal, the automatic disengagement reacts: avoiding the wire rupture and abnormal metal fatigue.

In the *Sensipast*, the miniature friction clutch is lodged in the mandrel itself. This conception allows it to be used on any kind of handpiece.

You can work stressless, for a perfect obturation!

Technical notes

The paste-filler is, in fact, a delicate instrument. Its active part consists of a steel wire of some hundredths of a millimetre. It is set in rotation by an apparatus of which the power is in no way related to the resistance of the wire that forms the spirals.

Sudden jamming often produces immediate breaking of classical paste-fillers. In such cases the automatic clutch system of the *Sensipast* acts to avoid this rupture by loosening the spirals of the driving system.

In addition, classical paste-fillers are often subjected to more or less considerable breakage, especially in the case of very curved canals, excessive bending near the mandrel, or due to the «to and from» movements the

operator gives to the instrument during the filling. Such abnormal fatigue finally leads to breakage of the wire.


In all such cases, *Sensipast* reacts by disengaging before the stresses pass the limit of elasticity of the metal.

In view of its high level of safety & long life, *Sensipast* can, moreover, be used advantageously in you everyday's filling.

But be aware! The *Sensipast* system is also sensible to a normal fatigue, so change your paste-filler from time to time.


The paste-filler with SENSITIVITY

Sensipast knows its capabilities and never surpasses them.

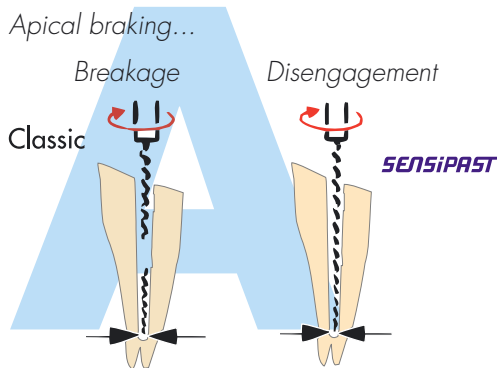

The paste-filler that is SAFE

The spirals of *Sensipast* block at the very moment where those of classical paste-fillers break.

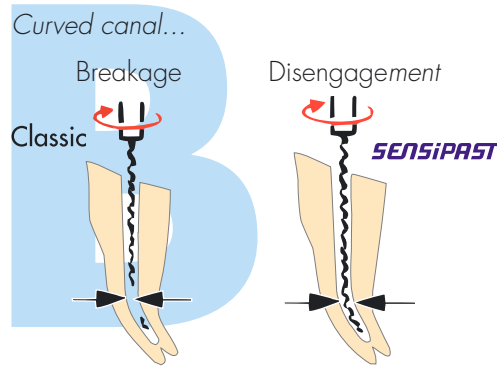

Availability - Ø ISO Sizes


25 30 35 40



- Per set of individual sizes - 4 pieces -
- Standard assortment ISO 25-40

Active part specification

L - mm
21 - 25 - 29

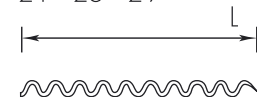


handle




The paste-filler that is EFFECIVE

Sensipast can be led into the canal more deeply and without fear of rupture in the apical region.

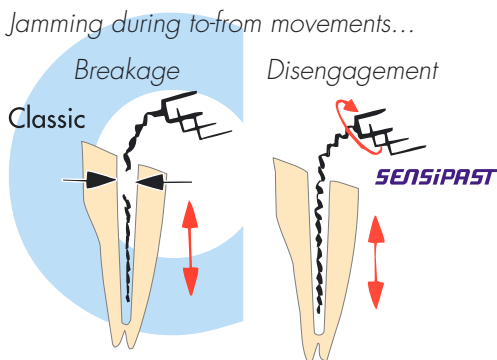

The paste-filler that is ECONOMICAL

The long life of *Sensipast*, greater than any other paste-filler, makes it a particularly economical instrument.

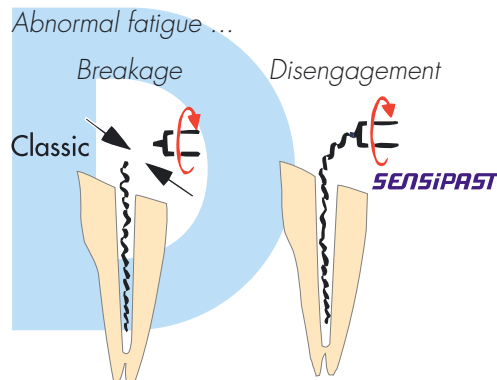

You can recognise *Sensipast* by its particular «nozzle». Its colour indicates the dimensions of the instrument according to the international ISO code.



SENSIPAST