

TOOLS TO KEEP SMILING

FKG All-Round SafetyMemoDisc



Better control of usage,
increased safety.

With *All-Round SMD*, safely record information on endodontic instrument utilisation.

You can control the fatigue of each rotary instrument, according to the complexity of the canal.

For hand instruments, you can count the **number of uses** or sterilisations.

The *All-Round SMD* system allows you to find your **own control method** and even new applications.

The *All-Round SMD* are **sterilisable** and stay on the instruments keeping you informed constantly.



The *All-Round SMD* are available in a new multi-purpose dispenser or in small bags of 100 discs.




FKG DENTAIRE
Swiss Dental Products




FKG REF. - 99.030.10.01D.AN

*A better control of risks,
for an optimum safety*

Thanks to the All-Round SMD, you can record essential information on rotary or manual instruments!



New instrument and its All-Round SMD

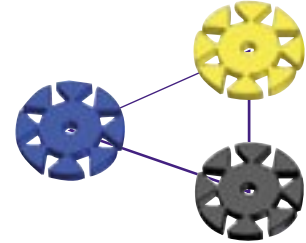

The All-Round SMD system allows you to find out your own control method and even new applications.

Each All-Round SMD has eight petals corresponding to a maximum of eight possible utilisations.

The All-Round SMD are sterilisable and stay on the instruments. You keep the information all along the file life.

With rotary instruments, remove from 1 to 8 petals, according to the instrument size/taper and the complexity of the canal.


A. determine root canal curvature:

- straight (Simple), radius ≥ 25 mm
- light curved (Medium), radius 25 until 11 mm
- apical hook/calcified (Difficult), radius 11 mm until 8 mm

B. Remove the required number of petals, i. e. Medium canal, radius from 25 until 11 mm:


- instrument $\varnothing 20 / 2\%$, remove 2 petals
- instrument $\varnothing 30 / 4\%$, remove 4 petals

C. The remaining petals indicate how many more times the file can be used.

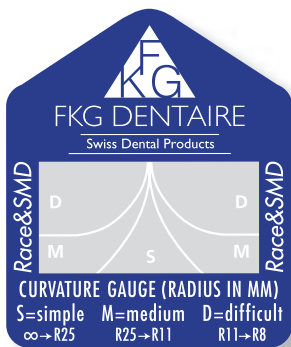

Measuring curvature gauge (scale 1:1)

For hand instruments, you can pull off one petal after each utilisation or sterilisation.

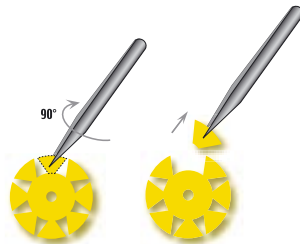

to remove petals, rotate & pull off



...usable



...to discard



All-Round SMD are available in small bags of 100 pieces or in a multi-purpose dispenser.

The dispenser comes with:

- model A: 100 All-round SMD
- model B: 70 All-round SMD & 70 Rubber stop (ISO colors)
- model C: 300 Rubber stop (ISO colors)

Special features of the dispenser

- Transparent recyclable plastic case
- Easy centering and insertion of the All-round SMD
- Positioning notch and scale for the insertion of the «Rubber stop»
- Improved grip zones

All-Round SMD



Suitable for instruments

No perforation		✓			\varnothing ISO 6-15
Perforation	\varnothing 0.5 mm	✓	✓	✓	\varnothing ISO 20-55
	\varnothing 0.9 mm	✓	✓		\varnothing ISO 60-90