# XP-endo<sup>®</sup> Finisher

Protocol  
Protocole  
Protokoll

[www.fkg.ch](http://www.fkg.ch)

## ► Protocol

800 rpm (800-1000)  
1 Ncm

1. XP-endo<sup>®</sup> Finisher (XP-F) must only be used only after preparing the canal to at least ISO 25.
2. In multirrooted teeth, begin with the largest canal. The canal should always contain irrigant. However, avoid filling the access cavity with irrigant before insertion of the file inside the canal. This is to prevent premature expansion of the file.
3. Fix the canal working length (WL) by using the plastic tube (a) to adjust the endo stop (b) (Fig. 1).
4. Cool the XP-F down inside the tube using a cold spray.
5. Put the XP-F in rotation mode and remove it from the tube by applying a lateral movement to ensure the XP-F remains straight (Fig. 2). Turn off the rotation.
 

5a. *The surface of the tube may be touched with the operators fingers only at its end, on FKG logo. This is to avoid warming the file.*

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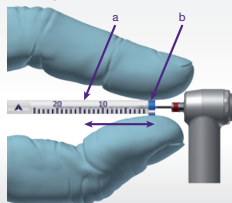
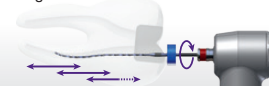
Figure/Abbildung 4

Figure/Abbildung 5

5b. If the file is straightened outside the tube use an alcohol soaked gauze to avoid warming of the file.

6. Insert the XP-F tip into the first canal of the tooth (Fig. 3). Once it is inside, turn on the rotation and slowly insert the file into the canal (Fig. 4). Add irrigant to the access cavity.

6a. In case of any difficulty inserting the file inside the canal of multirooted teeth, make sure to direct the tip of the expanded file towards the cusp related to the canal (e.g. MB cusp for the MB canals).

7. Use the XP-F for approximately 1 minute, using slow and gentle 7-8 mm longitudinal movements to contact the tip of the file on the full length of the canal (Fig. 5). Make parietal movements during the procedure. Avoid removing the file from the canal during this procedure.

8. After 1 minute, remove the XP-F from the canal while it is still in rotation.

9. Irrigate the canal to remove the suspended debris. A syringe tip 31G is recommended (or no bigger diameter than 30G).

To continue treatment inside a narrower canal of the same tooth previously shaped :

10. Clean the XP-F and place it back inside the measurement tube.

11. Begin again the procedure at step 3.

When the full cleaning of a tooth is completed, proceed to the next phase of treatment :

12. Obturate with gutta percha and sealer. *TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.*

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