XP-endo® Shaper

Protocol
Protocolle
Protocollo

www.fkg.ch

► Golden rules

800 rpm
1 Ncm

- Prior to using the XP-endo® Shaper, establish glide path to at least 15/.02.
- In multirooted teeth, begin with the largest canal.
- Never force the instrument.
- Irrigate copiously throughout the instrumentation protocol.

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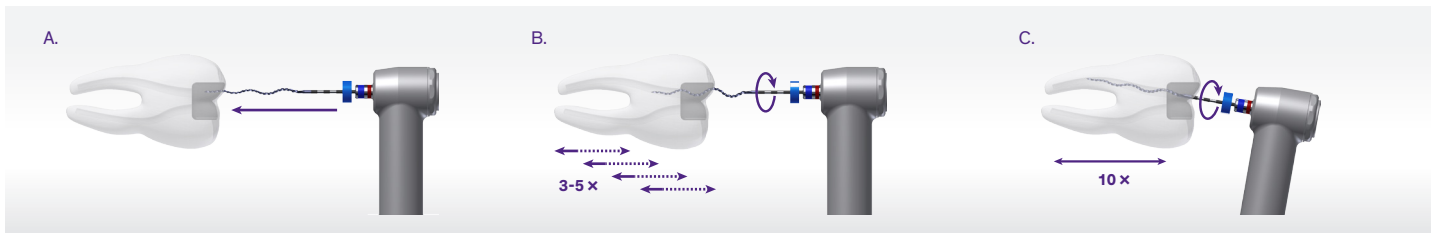
► Protocol

800 rpm (800-1000)
1 Ncm

1. Prior to using the XP-endo® Shaper, establish glide path to at least 15/.02. *In calcified/constricted canals or complex curvatures, a glide path to 10/.04 is recommended.*
2. Insert the tip of the XP-endo® Shaper into the canal (Fig. A), retract slightly and engage the handpiece in rotation mode. *The canal and the pulp chamber should always contain irrigant.*
3. Use gentle strokes to progress down to working length (WL) (Fig. B); disengage after each stroke. If WL is not reached in 3 to 5 strokes, stop, irrigate, recapitulate and proceed again with XP-endo® Shaper. *Never force the instrument and always keep it spinning and moving while in the canal.*
4. Once WL is reached, irrigate and work the instrument in long in and out gentle movements to WL for another 10 strokes (Fig. C). The final apical dimension is now at least 30/.04.
5. Irrigate the canal in order to eliminate suspended debris.
6. Confirm the final apical dimension with a 30/.04 gutta percha point.
7. If a larger preparation is required, use an appropriate Race instrument to obtain the desired final preparation.
8. Apply your regular irrigation protocol. *Final cleaning with XP-endo® Finisher is recommended.*
9. Obsolete with gutta percha and sealer. *TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.*

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