Instructions for use

- EN** see www.fkg.ch for additional information
- FR** informations supplémentaires sur www.fkg.ch
- DE** weitere Informationen auf www.fkg.ch

BT-Race

Endodontic instruments
Instruments endodontiques
Endodontische Instrumente

EN Golden rules

- Recommended speed : 800 rpm
Torque: 1.5 Ncm
- > Glide path should be established before using BT-Race sequence (min. size ISO 15/.02).
 - > All files should be used with long and gentle pecking motion (3-4 back and forth gentle strokes).
 - > Copious irrigation throughout the procedure.

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BT-Race sequence 1 blister includes/contient/enthält: 2 x (BT1, BT2, BT3)

For most cases

Speed : 800 rpm (600-1000 rpm)

1. After the coronal access is attained, the WL should be obtained with small hand files (ISO 06, 08, 10 or 15) depending the constriction of each canal.
2. A glide path should be performed with small stainless steel or NITI files up to ISO 15 before using BT-Race sequence.
3. Files of the entire sequence should be used to full WL before changing to the next file in the sequence. Per file, total working time in one canal should not exceed 10 seconds.
4. Use the sterile BT1 with a long and gentle pecking motion (3-4 back and forth strokes). If BT1 does not reach WL, clean the instrument, irrigate and repeat until the WL is achieved.
5. Recapitulate with K-File ISO 15 to remove the debris and keep the glide path open, irrigate.
6. Use sterile BT2 up to WL in the same manner as BT 1.
7. Recapitulate with K-File ISO15, irrigate.
8. Use sterile BT3 up to WL in the same way as BT 1.

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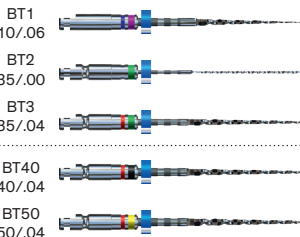
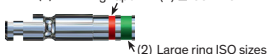
(1) Thin ring Taper (1) □.00 .04 .06

(2) Large ring ISO sizes

Mark on instruments with
Booster Tip

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For larger apical sizes

Speed : 800 rpm (600-800 rpm)

- BT3 is used for minimal biological apical preparation.
For larger apical preparations use BT40 or BT50.

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(DE)

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Sterile products - Use

1. Open the alveolus.
2. Extract the instrument (use of gloves imperative) and attach it to the contra-angle.
3. Use the instrument according to the protocol on front page.

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BT-Race instruments

are intended for single use only.

The reuse of instruments foretold as single use increase the risks of breakage and cross contaminations.

BT-Race instruments

are intended for single use only.

The reuse of instruments foretold as single use increase the risks of breakage and cross contaminations.

BT-Race instruments

are intended for single use only.

The reuse of instruments foretold as single use increase the risks of breakage and cross contaminations.

Caution

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

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