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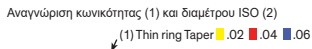
Taper (1) and ISO diameter (2) identification

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## SafetyMemoDisc (SMD)

Pull off petals to master metal fatigue:

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Speed : 600-800 rpm

Torque : 1.5 Ncm

- Long back and forth gentle strokes.
- Light touch, let the instrument work.
- Work 3-4 seconds in a row, get out, clean the blade and irrigate the canal.

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For canals with a severe apical curvature or S-shape

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator

1. Introduce ScoutRace (SR) 10/02 in rotation and reach WL;
2. Continue with SR 15/02 up to WL;
3. Finish glide path with SR 20/02 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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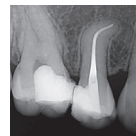
For narrow and/or calcified canals

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator.

1. Introduce Race 10/02 in rotation and reach WL;
2. Continue with Race 10/04 up to WL;
3. Finish glide path with Race 10/06 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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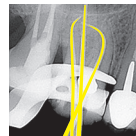
EN

## Sterilization protocol for reusable medical devices

### Non sterile products

#### First use

Carefully follow the rules herebelow  
**Cleaning and Sterilization**

### Sterile products

#### First use

1. Open the alevous
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice

#### Second use:

Carefully follow the rules herebelow  
**Second and following uses.**

### Cleaning

1. Manual cleaning with/without Ultrasonic assistance

Thorough cleaning allows efficient disinfection & sterilization.

2. Rinse
- Under distilled/demineralized water at least 1 min.
- Then dry the instruments.

3. Inspection - Check  
Sort out damaged or worn out instruments.
4. Packing  
Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

### Sterilization

5. According EN/ISO 17664 protocol  
Autoclave: 134°C/273°F, 2.2 bar during 18 min.

6. Storage  
Keep devices in sterilization wrap/pouch in a dry and clean environment.

 Always refer to manufacturer's instructions for cycle & duration

### Second and following uses

7. Pre-disinfection  
Plunge instruments after use in detergent solution and eventually brush them manually.

8. Rinse  
Under running water at least 1 min. Then dry the instruments.
9. Follow steps 1 - 6 above

 Complete operating protocol and warnings on [www.fkg.ch](http://www.fkg.ch)

### General information

#### Pre-/Disinfection DO NOT USE

- solutions containing:
- phenol (corrosion)
  - aldehyde (blood fixation)
  - di-/triethanolamines (corrosion)

**Sterilization with chemiclav or dry air devices** have not been validated by FKG Dentaire.  
Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

#### Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

A single patient use is recommended to avoid cross-contamination.

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