- (EN)** see www.fkg.ch for additional information
- (FR)** informations supplémentaires sur www.fkg.ch
- (DE)** weitere Informationen auf www.fkg.ch

FKG REF. 99.7AA.10.23A.XX - n°113 - 2021/12

RACE® EVO sequences

Description - Beschreibung

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(EN) General information

Parameters for use:

Recommended speed range: 800-1000 rpm
continuous rotation.

Torque: 1.5 Ncm.

Indications for use:

RACE® EVO endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

RACE® EVO instruments are designed for the glide path and shaping of root canals.

Contraindications:

Nickel-Titanium instruments must not be used on individuals with a known allergic sensitivity to these metals.

Adverse reactions: None known.

Noted and precautions:

- Take several radiographs from different angles to determine the anatomy of the root canals (length, width and curvature).
- The number of canals to be treated with a RACE® EVO instrument is limited to a maximum of 8 root canals.
- Irrigate the canal thoroughly and frequently after the RACE® EVO instrument is extracted.
- Regularly inspect the RACE® EVO instrument during use, and discard if it shows any signs of wear (for example: straightening).
- If the RACE® EVO instrument is not progressing easily, remove it from the canal, clean it, inspect its cutting edges, then irrigate the canal and recapitulate with an ISO 010 hand file.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.
- Visit the FKG Dentaire SA website (www.fkg.ch) for all information relating to the applicable cleaning and sterilization methods.

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1. Create straight-line coronal and radicular access.
2. Use an ISO 010 hand file to explore the canal.
3. Determine the working length using a radiograph and/or an apex locator.
4. Use the RACE® EVO instruments in the order predefined by the chosen sequence. Start with glide path, followed by shaping of the canal.
5. RACE® EVO instruments are to be used with gentle strokes of 2-3 mm, applying very light apical pressure and allowing the instrument to progress passively along the canal. After 3 strokes, remove and clean the instrument, then irrigate; recapitulate with an ISO 010 hand file.
6. Use the mechanised RE1 instrument to perform glide path until the working length is reached.
7. Start the shaping procedure, applying your usual irrigation/ disinfection protocol.
8. Use the RE2 until the working length is reached.
9. Use the RE3 until the instrument can be inserted to its working length. *If necessary, a brushing motion may be applied to remove coronal interferences and/or ensure uniform shaping of irregular canals.* Once the working length has been reached, remove the instrument to prevent over-enlargement of the foramen.
10. If the preparation is not sufficient for the anatomy of the canal being treated, then continue shaping the canal using the larger sizes of RACE® EVO instruments.

We recommend that 40/.04 or 50/.04 instruments are used for shaping larger canals, following the RACE EVO 4% sequence. We recommend that 30/.06 or 35/.06 instruments are used for shaping larger canals, following the RACE EVO 6% sequence.

Once all the canals have been shaped, proceed with the cleaning of the canals. For an optimal result, the use of XP-endo® Finisher is recommended.

Once the cleaning of the canals is completed, proceed to the next phase of treatment:

11. Obturate with gutta-percha and sealer. TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.

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