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ScoutRace & ISO 10

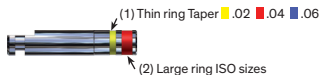
Description - Beskrivelse - Kuvaus

Taper (1) and ISO diameter (2) identification

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SafetyMemoDisc (SMD)

Pull off petals to master metal fatigue:

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Speed : 600-800 rpm

Torque : 1.5 Ncm

- Long back and forth gentle strokes.
- Light touch, let the instrument work.
- Work 3-4 seconds in a row, get out, clean the blade and irrigate the canal.

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For canals with a severe apical curvature or S-shape

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator

1. Introduce ScoutRace (SR) 10/.02 in rotation and reach WL;
2. Continue with SR 15/.02 up to WL;
3. Finish glide path with SR 20/.02 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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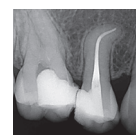
For narrow and/or calcified canals

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator

1. Introduce Race 10/.02 in rotation and reach WL;
2. Continue with Race 10/.04 up to WL;
3. Finish glide path with Race 10/.06 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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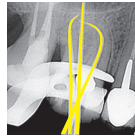
EN Sterilization protocol for reusable medical devices

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Non sterile products

First use
Carefully follow the rules herebelow
Cleaning and Sterilization

Sterile products

First use
1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice
Second use:
Carefully follow the rules herebelow
Second and following uses.

Cleaning

1. Manual cleaning with/without Ultrasonic assistance
Thorough cleaning allows efficient disinfection & sterilization.
2. Rinse
Under distilled/demineralized water at least 1 min.
Then dry the instruments.

3. Inspection - Check
Sort out damaged or worn out instruments.
4. Packing
Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

Sterilization

5. According EN/ISO 17664 protocol
Autoclave: 134°C/273°F, 2,2 bar during 18 min.

6. Storage
Keep devices in sterilization wrap/pouch in a dry and clean environment.

 Always refer to manufacturer's instructions for cycle and duration

Second and following uses

7. Pre-disinfection
Plunge instruments after use in detergent solution and eventually brush them manually.

8. Rinse
Under running water at least 1 min. Then dry the instruments.
9. Follow steps 1 - 6 above

 Complete operating protocol and warnings on www.fkg.ch

General information

Pre-/Disinfection DO NOT USE
solutions containing:
- phenol (corrosion)
- aldehyde (blood fixation)
- di-/triethanolamines (corrosion)

Sterilization with chemiclav or dry air devices have not been validated by FKG Dentaire.
Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

Caution:
This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

A single patient use is recommended to avoid cross-contamination.

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