
- EN** see www.fkg.ch for additional information
- FR** informations supplémentaires sur www.fkg.ch
- DE** weitere Informationen auf www.fkg.ch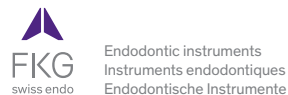

FKG REF. - 99.000.10.02A.XX - n°101 - 2020/01

EN

Sterilization protocol for reusable medical devices

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General advice:

Instruments stamped  are intended for single use only

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(EN) Sterilization protocol for reusable medical devices

Non sterile products

First use
Carefully follow the rules herebelow
Cleaning and Sterilization

Sterile products

First use
1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice
Second use:
Carefully follow the rules herebelow
Second and following uses.

Cleaning

1. Manual cleaning with/without Ultrasonic assistance
Thorough cleaning allows efficient disinfection & sterilization.
2. Rinse
Under distilled/demineralized water at least 1 min.
Then dry the instruments.

3. Inspection - Check
Sort out damaged or worn out instruments.
4. Packing
Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

Sterilization

5. According EN/ISO 17664 protocol
Autoclave: 134°C/273°F,
2.2 bar during 18 min.

6. Storage
Keep devices in sterilization wrap/pouch in a dry and clean environment.

 Always refer to manufacturer's instructions for cycle & duration

Second and following uses

7. Pre-disinfection
Plunge instruments after use in detergent solution and eventually brush them manually

8. Rinse
Under running water at least 1 min.
Then dry the instruments.
9. Follow steps 1 - 6 above

 Complete operating protocol and warnings on www.fkg.ch

General information

Pre-/Disinfection DO NOT USE
solutions containing:
- phenol (corrosion)
- aldehyde (blood fixation)
- di-/trietanolamines (corrosion)

Sterilization with chemiclav or dry air devices have not been validated by FKG Dentaire.
Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

 200°C (392°F)

A single patient use is recommended to avoid cross-contamination.

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