## Instructions for use

- EN** see www.fkg.ch for additional information
- FR** informations supplémentaires sur www.fkg.ch
- DE** weitere Informationen auf www.fkg.ch

### XP-endo® Finisher R

Endodontic instruments  
Instruments endodontiques  
Endodontische Instrumente

FKG REF. 99\_X00-10.04A\_XX - n°107 - 2020/01

## XP-endo® Finisher R

### Description - Beschreibung

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Recommended Speed: 800 rpm

Torque: 1 Ncm

- In multirooted teeth, begin with the largest canal.
- The access cavity must be filled with solvent before inserting the XP-endo Finisher R into the canal.
- Work the entire length of the canal for 30 seconds with solvent, 30 seconds with irrigant and 15 seconds with EDTA.

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## General information

### Indications for use:

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

XP-endo Finisher R is used for endodontic retreatments, to finish off the filling removal phase.

### Contraindications:

This product contains nickel, and must not be used on individuals with a known allergic sensitivity to this metal.

**Adverse reactions:** None known

### Warnings and precautions:

- Single use instruments must not be retreated for subsequent use. That could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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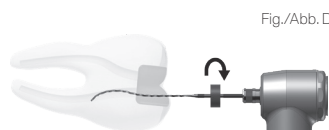
Fig./Abb. D

Speed : 800 rpm (800-1000 rpm)  
Torque : 1 Ncm

1. Use XP-endo Finisher R (XP-FR) only after a shaping procedure of at least #30.
2. Apply a drop of solvent in the canal.
3. Remove an XP-FR from the sterile blister pack, and place it in a contra-angle handpiece.
4. Define the working length using the graduated tube. (Fig. A)
5. Cool the XP-FR with a cooling spray via the tube.
6. Switch the XP-FR to rotate mode, and press against the side walls as you remove it from the tube, to ensure that the file is straight (Fig. B). Then deactivate rotate mode.
  - 6a. Touch only the end of the tube, on the FKG logo, in order to prevent the file from heating up.
7. Insert the straightened XP-FR into the canal to be treated (Fig. C), and then switch it to rotate mode (Fig. D).
  - 7a. In case of difficult access, the instrument can be pre-curved.
8. Use the XP-FR for 30 seconds, applying slow, gentle longitudinal movements of 7-8 mm, so as to cover the entire length of the canal. Press the instrument against the canal walls during the procedure. Avoid removing the instrument from the canal.
9. Remove the XP-FR from the canal while it is rotating. Deactivate rotate mode.
10. Repeat the job, making sure to alternate the solutions (30 seconds with irrigant, then 15 seconds with EDTA) in order to eliminate the suspended debris.

Between each solution, clean off the debris remaining on the instrument using an alcohol soaked gauze.

To continue the treatment in a narrower canal on the same tooth:

11. Clean the XP-FR and put it back in its tube.
12. Restart the procedure from step 4.

Once you have finished fully cleaning a tooth:

13. Dry the canals and fill them using an inert cone (e.g. gutta percha) and a sealer (e.g. TotalFill™).

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