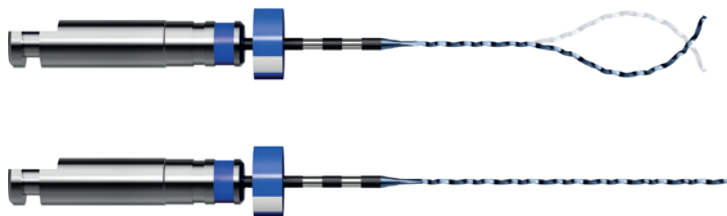
# XP-endo<sup>®</sup> Finisher R

Protocolo  
Protocolo  
Protocol

STERILE R

## ► Reglas de oro

800 rpm  
1 Ncm

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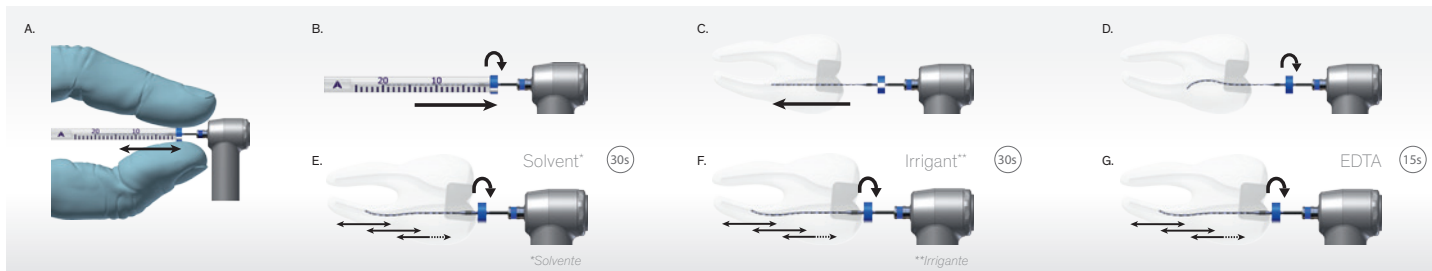
## ► Golden rules

800 rpm  
1 Ncm

- In multirrooted teeth, begin with the largest canal.
- The access cavity must be filled with solvent before inserting the XP-endo<sup>®</sup> Finisher R into the canal.
- Work the entire length of the canal for 30 seconds with solvent, 30 seconds with irrigant and 15 seconds with EDTA.

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## ► Protocol

800 rpm (800-1000)  
1 Ncm

1. Apply a drop of solvent in the canal.
2. Define the working length using the graduated tube (Fig. A).
3. Cool the XP-endo Finisher R (XP-FR) with a cooling spray via the tube.
4. Switch the XP-FR to rotate mode, and press against the side walls as you remove it from the tube, to ensure that the file is straight (Fig. B). Then deactivate rotate mode.
5. Insert the straightened XP-FR into the canal to be treated (Fig. C), and then switch it to rotate mode (Fig. D).
6. Use the XP-FR for 30 seconds, applying slow, gentle longitudinal movements of 7-8 mm, so as to cover the entire length of the canal (Fig. E). Press the instrument against the canal walls during the procedure. Avoid removing the instrument from the canal.
7. Remove the XP-FR from the canal while it is rotating. Deactivate rotate mode.
8. Repeat the job, making sure to alternate the solutions (30 seconds with irrigant, then 15 seconds with EDTA) in order to eliminate the suspended debris (Fig. F-G). Between each solution, clean off the debris remaining on the instrument using an alcohol soaked gauze.