Instructions for use

- EN** see www.fkg.ch for additional information
- FR** informations supplémentaires sur www.fkg.ch
- DE** weitere Informationen auf www.fkg.ch

XP-endo® Finisher

Endodontic instruments
Instruments endodontiques
Endodontische Instrumente

FKG REF. 99.X00.10.01A.XX - n°108 - 2021/12

XP-endo® Finisher

Description - Beschreibung

25/00

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Golden rules

Speed : 800-1000 rpm
Torque : 1 Ncm

- XP-endo® Finisher should be used only after canal preparation to at least ISO 25.
- In multirrooted teeth, begin with the largest canal.
- Work along the entire length of the canal for approximately 1 minute.

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General information

Indications for use :

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

The XP-endo® Finisher is used as a final cleaning instrument in root canal treatments.

Contraindications :

This product contains nickel, and must not be used on individuals with a known allergic sensitivity to this metal.

Adverse reactions : None known

Warnings and precautions :

• Single use instruments must not be retreated for subsequent use. That could compromise their integrity.

• Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).

• When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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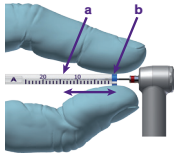
Figure / Abbildung 1

Speed: 800-1000 rpm
Torque: 1 Ncm


1. XP-endo® Finisher (XP-F) must only be used only after preparing the canal to at least ISO 25.
2. In multirooted teeth, begin with the largest canal. The canal should always contain irrigant.
3. Fix the canal working length by using the plastic tube (a) to adjust the endo stop (b) (Fig. 1).
4. Place the XP-F into the canal by lining up the tip facing the cusp, insert slightly into the canal (Fig. 2). Then start the motor and slowly thread the XP-F into the canal.
5. Use the XP-F for 1 minute (approximately 60 strokes), using slow and gentle 7-8 mm longitudinal movements to contact the full length of the canal (Fig. 3). Make small multidirectional movements and avoid removing the file from the canal.
6. Irrigate the canal to remove the suspended debris. *A syringe tip 31G is recommended (or no bigger diameter than 30G).*

Once the cleaning of the canals is completed, proceed to the next phase of treatment:

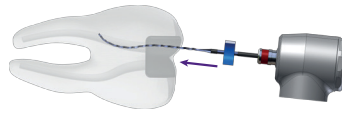7. Obtrate with gutta percha and sealer. *TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.*

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