



D-Race 1

↻ 800-1000RPM – 1.5Ncm

XP-endo® Rise Shaper  
30/.04

↻ 1000-2500RPM – 1Ncm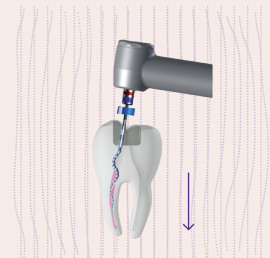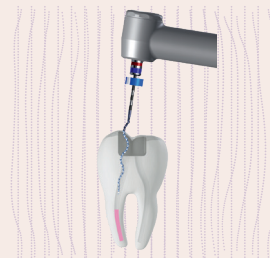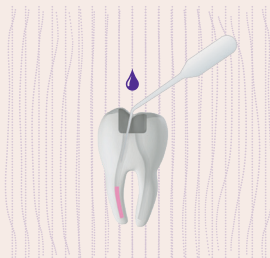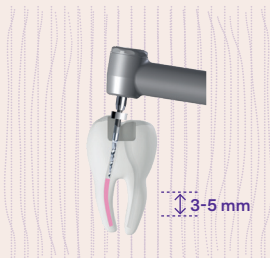

### EN

- Create a start in the coronal portion of the gutta-percha (3-5mm) using D-Race 1.

- Place one drop of solvent in the coronal space created and wait for at least 1 minute.

- Place the tip of XP-endo® Rise Shaper into the prepared space in the gutta-percha, disengage slightly and start the motor at 1000-2500 rpm.

- Use pecking motion until the XP-endo® Rise Shaper's tip engages the gutta-percha. Then use light pressure to help advance the XP-endo® Rise Shaper down the canal until working length.

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



800-1000RPM – 1Ncm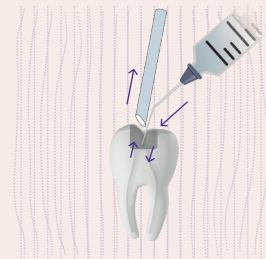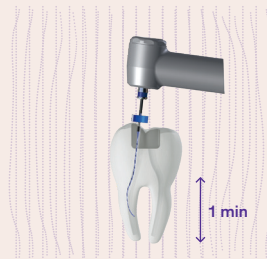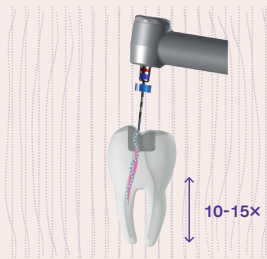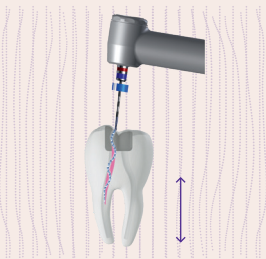

### EN

• Allow the XP-endo® Rise Shaper to gently corkscrew around the gutta percha and tease out large strands of gutta percha using long gentle strokes.

• After the removal of the gutta percha, measure the working length (WL), irrigate and fill the pulp chamber and canals with NaOCl. Use the XP-endo® Rise Shaper for 10-15 additional long gentle strokes to WL.

• Use the Finisher R for at least 1 minute (longer, if debris is still coming out of the canal), using slow and gentle longitudinal movements to contact the full length of the canal.

• Use copious NaOCl irrigation and suction to remove the gutta percha and sealer tags prior to complete the treatment.

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