Instructions for use

- (EN)** see www.fkg.ch for additional information
- (FR)** informations supplémentaires sur www.fkg.ch
- (DE)** weitere Informationen auf www.fkg.ch

XP-endo® Shaper

Endodontic instruments
Instruments endodontiques
Endodontische Instrumente

FKG REF. 99.X00.10.05A.XX - n°108 - 2021/12

XP-endo® Shaper

Description - Beschreibung

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Golden rules

Speed : 800-1000 rpm
Torque : 1 Ncm

- Prior to using the XP-endo® Shaper, establish glide path to at least 15/.02.
- In multirrooted teeth, begin with the largest canal.
- Never force the instrument.
- Irrigate copiously throughout the instrumentation protocol.

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General information

Indications for use :

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

XP-endo® Shaper is used for endodontic treatments.

Contraindications :

This product contains nickel, and must not be used on individuals with a known allergic sensitivity to this metal.

Adverse reactions : None known

Warnings and precautions :

- Single use instruments must not be retreated for subsequent use. That could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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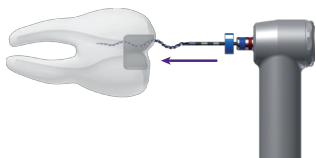
Figure / Abbildung 1

Speed: 800-1000 rpm
Torque: 1 Ncm

1. Prior to using the XP-endo® Shaper (XP-S), establish glide path to at least 15/.02. In calcified/constricted canals or complex curvatures, a glide path to 10/.04 is recommended.
2. In multirooted teeth, begin with the largest canal. The canal should always contain irrigant.
3. Insert the tip of the XP-S into the canal until resistance (Fig. 1), retract (tip loose) and start the motor.
4. Use long gentle strokes to progress down to working length (WL) (Fig. 2). If WL is not reached in 5 strokes, stop, irrigate, recapitulate and proceed again. Do not use pecking motion! Never force the instrument and always keep it spinning and moving while in the canal.
5. Once WL is reached, irrigate and work the instrument for 15 additional long gentle strokes to WL (Fig. 3).
6. Irrigate the canal in order to eliminate suspended debris.
7. Choose a gutta percha point that is tight and seats at the correct WL. If 30/.04 is desired and does not fit, work the XP-S for an additional 5-10 strokes.


Once all the canals have been shaped, proceed with the cleaning of the canals. For an optimal result, the use of XP-endo® Finisher is recommended.

Once the cleaning of the canals is completed, proceed to the next phase of treatment:

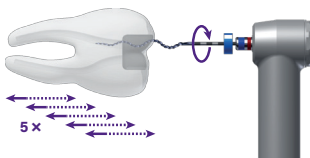8. Obturate with gutta percha and sealer. TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.

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