


FKG Dentaire world premiere:

1 - Indicator of number of uses on endo hand instruments.

The *SafetyMemoDisc (SMD)* system, originally on FKG RaCe rotary NiTi instruments, is a success. FKG Dentaire applied the same concept to stainless steel hand files and we call it **SafetyMemoGrip (SMG)**.

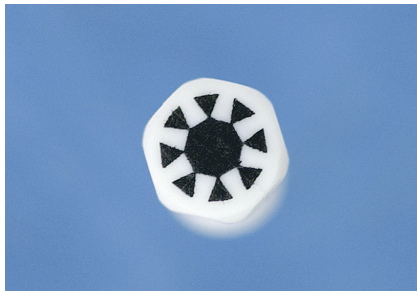

SafetyMemoGrip at the back of the new handle.




The information on the number of uses/sterilisations is recorded all along the file life. Practitioners will scratch one or more petals off after each use, depending on the effort on the blade and the canal curvature. The remaining petals indicate how many more times the instrument can be used.

2 - New ergonomic handle

At the same time, we have developed a new ergonomic handle, featuring:

- an improved grip - bean-shape;
- a larger diameter at the back end, for an easier, more comfortable and safer pull;
- improved control of feather-touch rotation - balanced force.

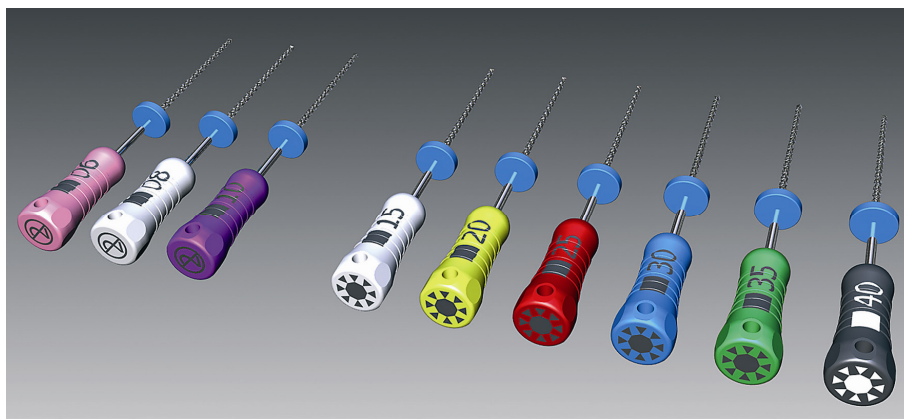

Availability

- Assortments (ISO 6-25 and 15-40)
- Individual sizes (6 pces/box)

		Ø ISO								
		6	8	10	15	20	25	30	35	40
K-file Standard	art. 154	•	•	•	•	•	•	•	•	•
K-file Flex	art. 554		•	•	•	•	•	•	•	•
H Std/Flex	art. 153/553		•	•	•	•	•	•	•	•
Reamer Standard	art. 160	•	•	•	•	•	•	•	•	•
Reamer Flex	art. 560				•	•	•	•	•	•

Safetytip as a standard